


Prior to rinse back, prepare all necessary supplies:




- Sterile scissors
- Irrigation syringe
- 500 mL bag of saline
- Disinfectant solution
- Hemolung Rinse Back Kit

To enter Rinse Back mode on the Controller:


- First press the Sub Menu function key
- Next press the End Therapy key
- Then press Continue to begin the on screen instructions




WARNING: Once rinse back is initiated, pump will stop and user cannot restart therapy without power cycling the Controller.

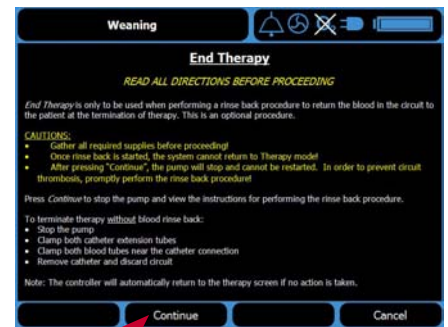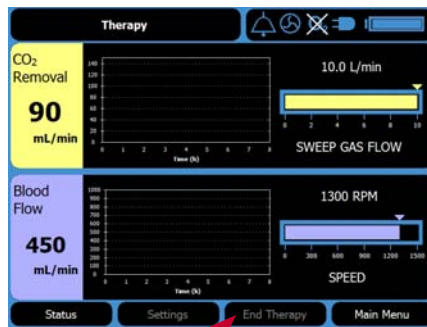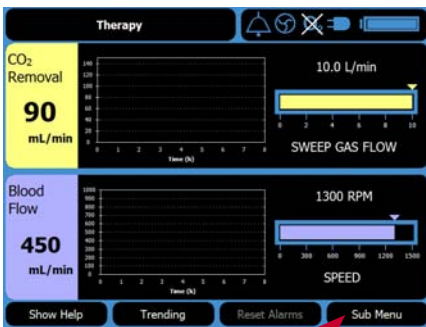


1 Close Blue Clamp and Spike Saline Bag

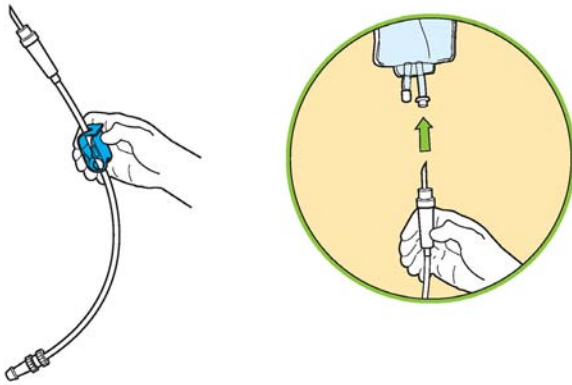


2 Hang the Saline Bag and Prime Drip Chamber

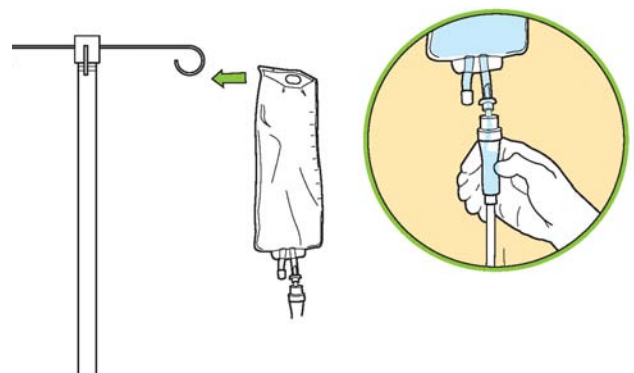


3 Clamp Blue Tubing and Lumen

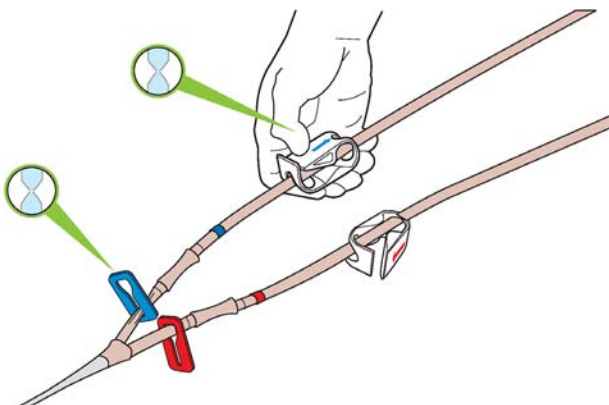


4 Sterilize FROM PATIENT (Blue) Tubing

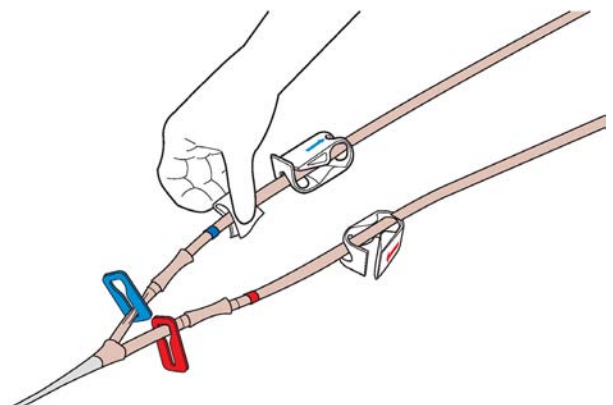


5 Cut FROM PATIENT (Blue) Tubing

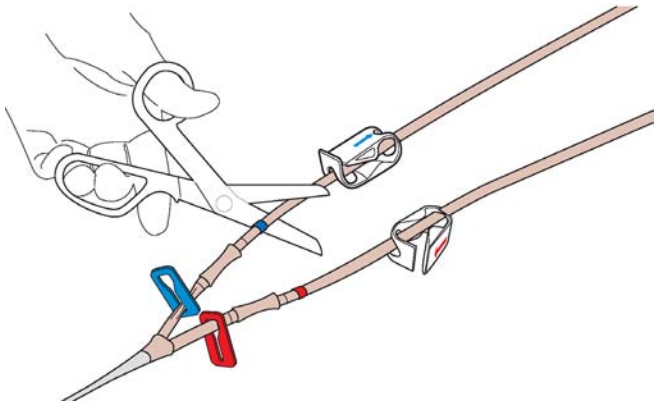


6 Connect FROM PATIENT (Blue) Tubing to IV Tube

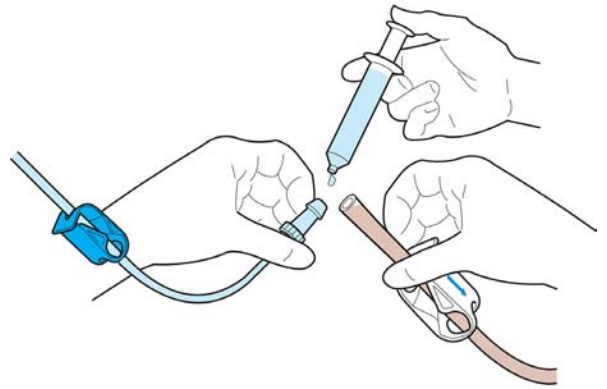


7 Release Blue Clamps

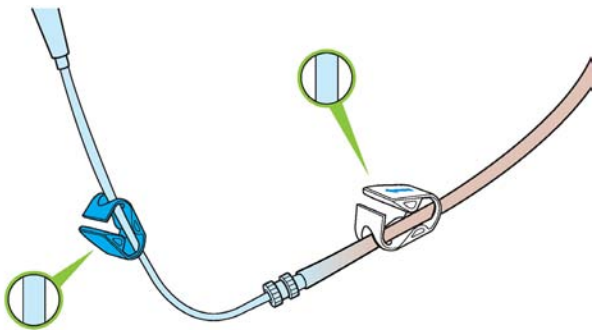


8 Do not exceed 250 cc of saline for flush back

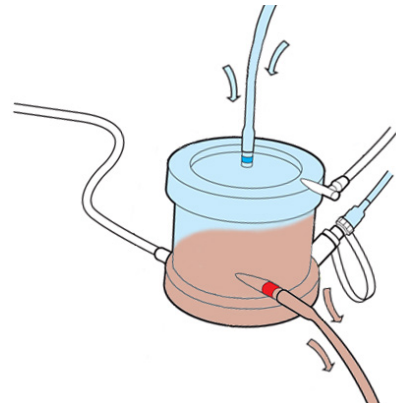

9 Once Blood is Returned, Close Blue Clamps

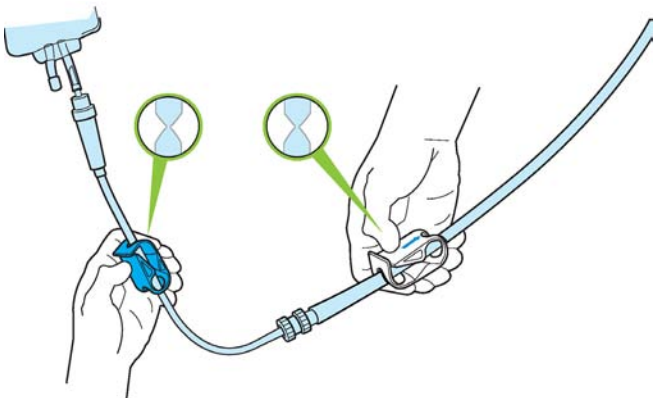

10 Clamp Red Tubing and Remove Catheter

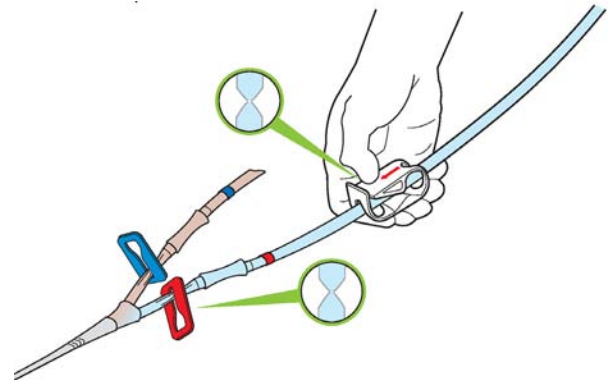

11 Dispose of Catheter and blood circuit following hospital procedures for biological wastes.



WARNING: If circuit thrombosis is suspected, do not rinse back blood to patient at conclusion of therapy or when replacing Cartridge.