


Before **Setup**, collect the following supplies:

- One 500 mL or 1000 mL bag/bottle of priming solution (0.9% NaCl with 1 U/mL heparin added)
- One 500 mL or 1000 mL bag of normal saline for seal flush

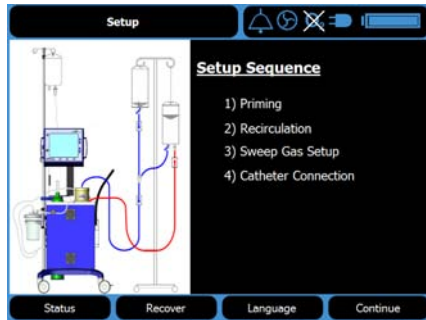


To begin:


- Plug in Controller
- Turn power switch ON
- Press Continue to set up New Circuit (press Recover to resume therapy)

Prime Circuit following these steps:


### 1 Remove Old Soda Lime

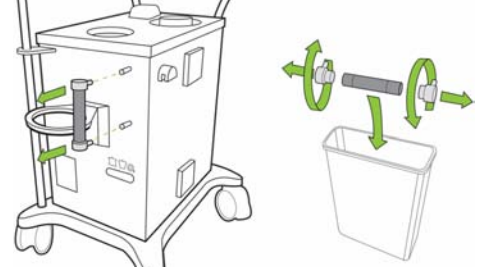


### 2 Assemble and Attach New Soda Lime Column

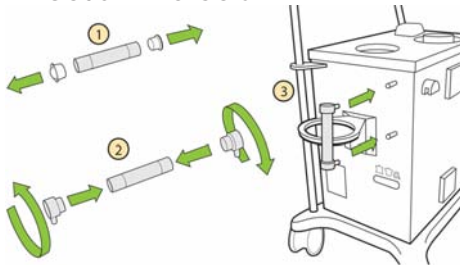


### 3 Hang the Saline for Seal Flush

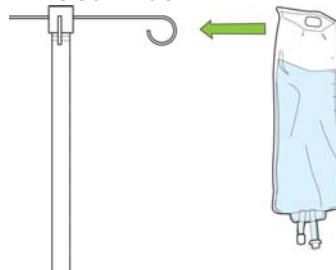


### 4 Set up Seal Flush

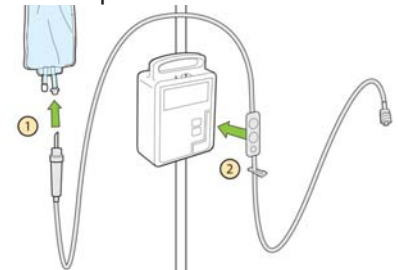


### 5 Connect IV Administration Set to Cartridge

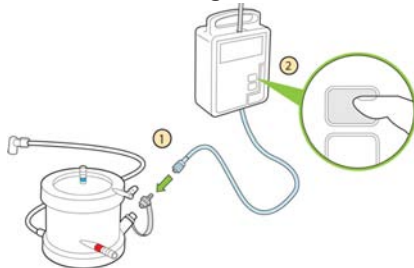


### 6 Prepare Priming Solution

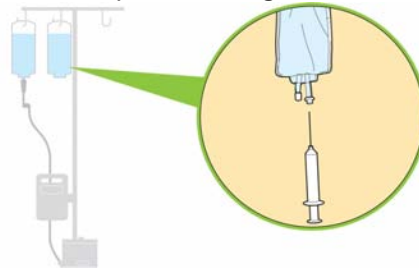


### 7 Hang Recirculation Bag

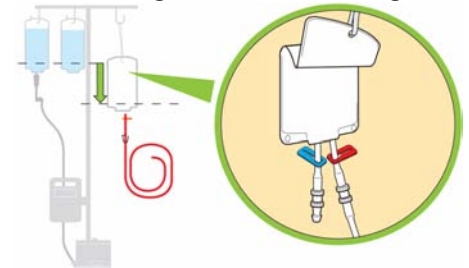


### 8 Connect Red Tube to Cartridge

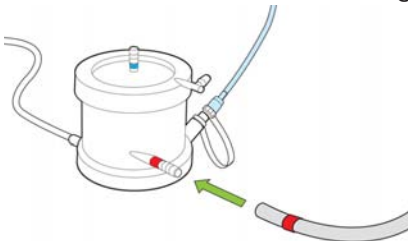


### 9 Spike the Priming Solution

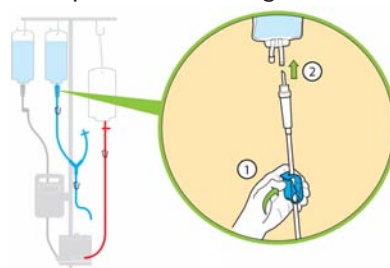


### 10 Connect Short Blue Tube to Recirculation Bag

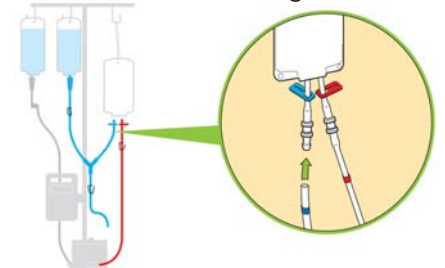


### 11 Connect Blue Tube to Cartridge

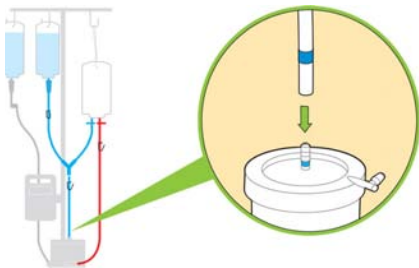


### 12 Fill the Drip Chamber

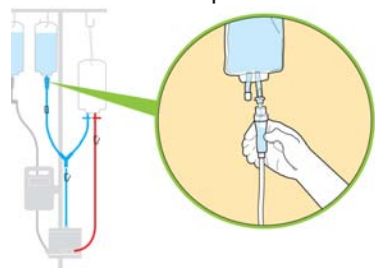


### 13 Close Blue Slide Clamps on Recirculation Bag

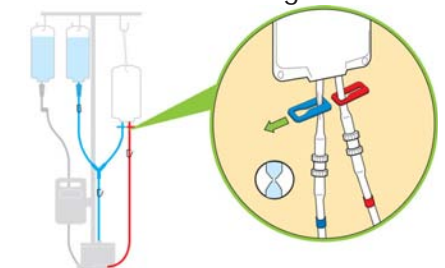


14 Prime the Cartridge

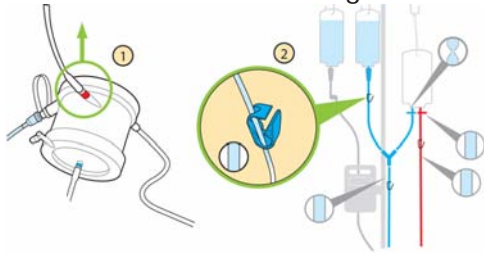


15 Use Gravity to Fill Cartridge

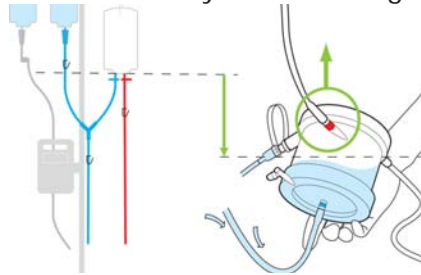


16 Fill the Cartridge

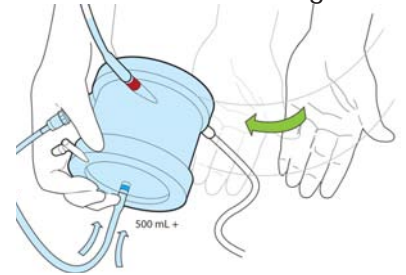


17 Close Blue Clamp on Priming Spike

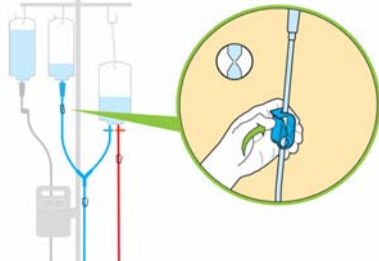


18 Open Blue Clamp on Recirculation Bag

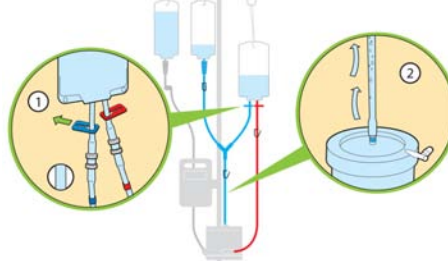


19 Check System for Air

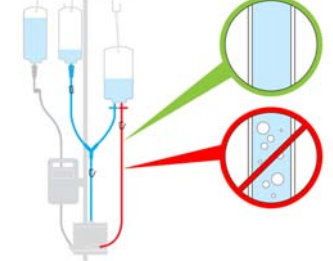

20 Apply Petroleum Jelly, Install Cartridge and Tubing

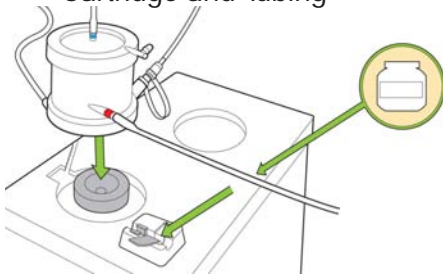


21 Assemble the Vacuum Canister

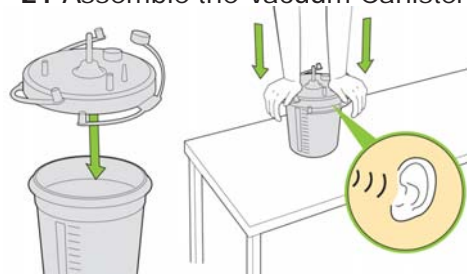


22 Close Unused Ports and Attach Vacuum Tubing




23 Connect Vacuum Canister to Controller

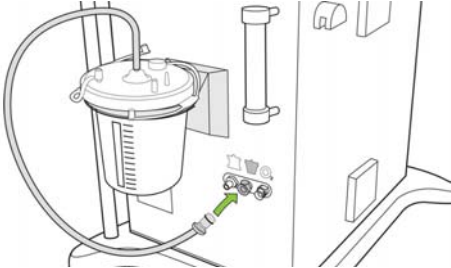


24 Install Silicone Tube in Purge Valve and Connect to Vacuum Canister

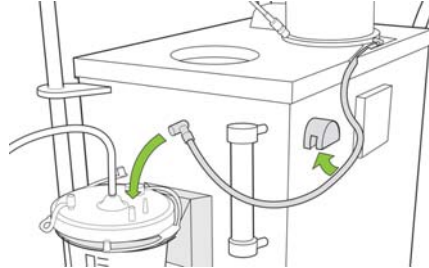

25 Connect Sweep Gas Supply Tube

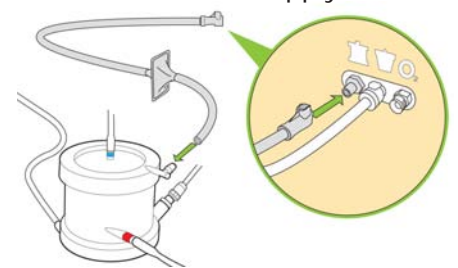


26 Check Tubing Connections and Check for Air

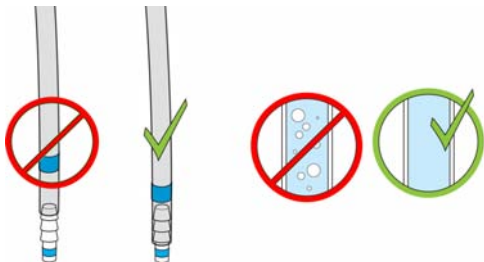


27 Start Recirculation




28 Allow System to Self-Test

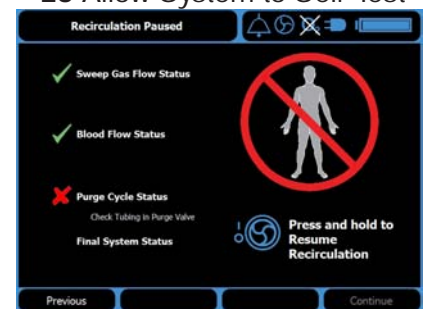

29 Successful Completion

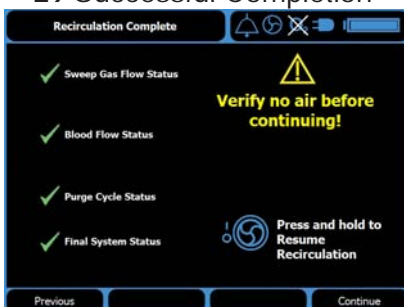

30 Select Sweep Gas Source

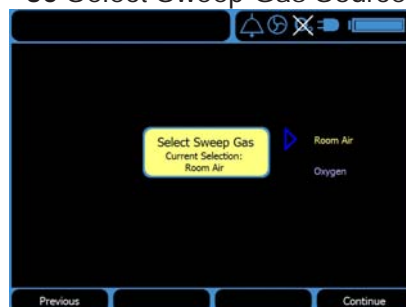

31 No Immediate Action Required on Controller

