

Setting up the Body Guard 323 Color Vision™ Infusion Pump


1 Hang and spike the saline bag, then slowly fill the drip chamber about 1/2 of the way full.



2 Plug in and power on the pump by holding down the ON/OFF button.




3 Open door and install line in pump from left to right. Make sure key is inserted correctly, then close door.

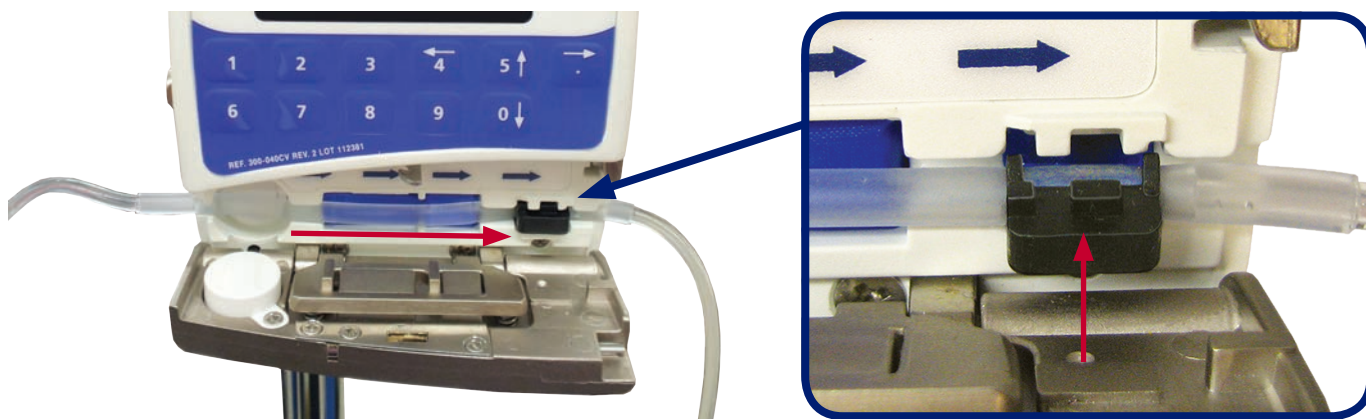

4 Press the Prime button, then immediately press Start. Enter 20 mL for priming volume, and press Start to begin.




5 Connect check valve to seal flush port on Cartridge, then connect IV administration set to check valve and press Start to begin.

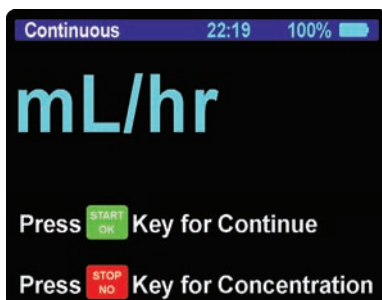

Concentration can be entered as 0.0 as it is not applicable to saline.


6 Enter 30 mL for Rate then press Start.

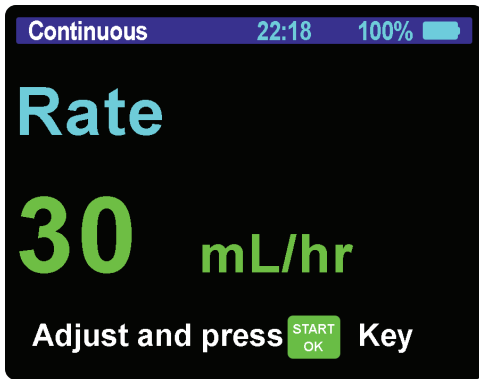


7 Enter actual volume of IV bag for Volume then press Start.




8 Review on-screen info then press Start.

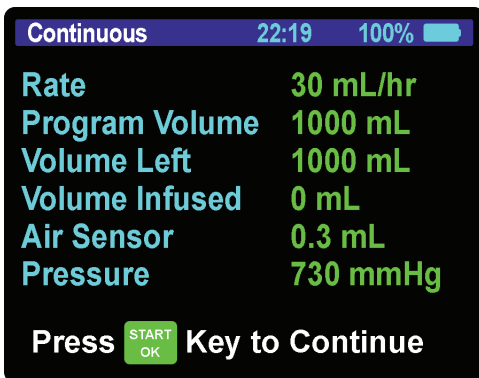

9 Press Start when ready to begin.

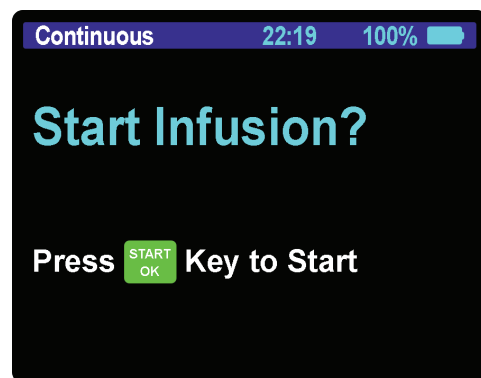

10 Infusion will start and rate will be displayed. Confirm green LED is blinking.

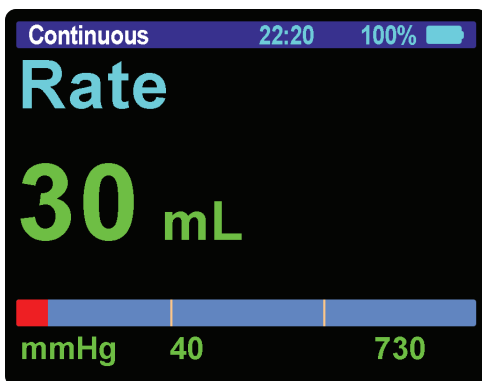

i Refer to the BodyGuard 323 Color Vision™ Manual for further instructions.

i Ensure pump is set to Continuous Program.